


## HAND CONTROL SETUP/RESET


### ZK3100-C5M Setup/Reset

**Procedure:** Press and hold all three buttons simultaneously (see photo) until the chair stops moving. Continue to hold all three buttons simultaneously for an additional three seconds.

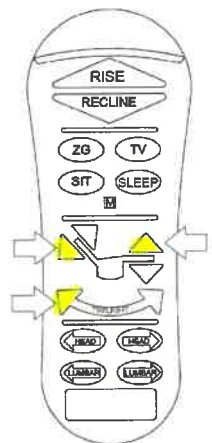


### ZK3200-HC Setup/Reset

**Procedure:** Press and hold both buttons simultaneously (see photo) until the chair stops moving. Continue to hold both buttons simultaneously for an additional three seconds.

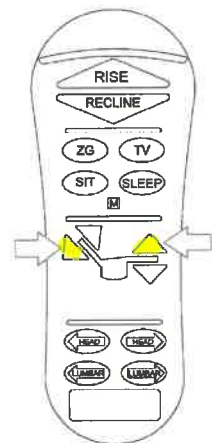

## Final Setup/Reset Position

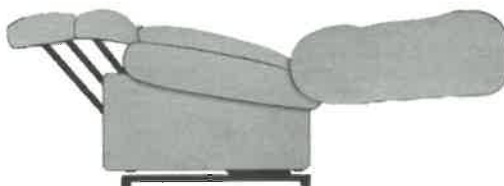

- The control system relies on sensors and memory to function properly.
- From time to time, the control system may need to be reset.
- This may be due to normal circumstances such as disconnecting power abruptly, power surges, or electrical outages.
- If the movement of the chair does not feel correct, resetting the system may resolve the issue.

## Troubleshooting

The setup of the chair requires the steps to be conducted as stated. If the chair does not operate correctly one of the following may have occurred:

- The incorrect buttons were used during setup.
- The buttons were released prior to the chair reaching the final setup position.
- Power disconnected or lost during setup.

The position of the chair when using the setup procedure will not affect operations. If any of these issues occurred and the chair does not seem to operate correctly, please repeat the Setup Procedure.