

Instructions to Sync a Remote Control

Turn off the scooter and do the following:

- 1) Make sure the scooter is in the unfolded position
- 2) Press and hold the "A" Button on the remote control
- 3) Remove the scooter battery for 5 seconds while continuing to hold the "A" Button
- 4) Replace the scooter battery while continuing to hold the "A" button and count to 10 seconds.
- 5) Let go of the "A" button. Then press and hold the "A" button for the scooter to fold.
- 6) The scooter may start to fold by itself while holding the "A" button down for 10 seconds.
- 7) Repeat steps 2-5 if the scooter does not fold

Note: Only one remote can be synced with a scooter at a time